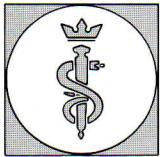
AESCULAP®

VARADY
Mikrochirurgische Phlebextraktion
[Microsurgical Phlebextraction acc.](#)
Flebexstracción microquirúrgica

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Microsurgical Phlebectomy acc. to VARADY

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Varices appear in various forms; applying therefore on principal a certain treatment scheme – such as stripping of the V. s. magna resp. parva – is surely not correct. Branch varicosis is often the cause with many patients, but not in every case there exists a trunk insufficiency.

The best medical and cosmetic results are achieved here by using a new microsurgical operation technique for isolated removal of the branches. Especially where female patients are concerned then – apart from the medical success – the cosmetic results are also of great importance (operation scars are avoided). With stripping or crossectomy it is also recommended to remove the numerous branches by applying the technique mentioned.

This method combines the advantages of operation and obliteration under simultaneous avoidance of their disadvantages.

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Advantages

- Minimum operation strain due to infiltration anaesthesia
- Lowest risk of infection, due to tiny stab incisions
- No pigmentation, no painful indurations as occur after obliteration
- No painful squeezing of the convolutions
- Hardly any ligatures
- No cicatrisation
- No formation of imprints as with other hook methods
- No after-bleeding as there is no operation on the saphena trunks
- After a short follow-up treatment the patients are able to return to work
- No repeated after-treatment as is the case after obliteration
- In most cases only an outpatient treatment is necessary – or else a short hospitalisation only
- Early mobilization, the most important prophylaxis against thrombosis, is possible immediately after the treatment as the patients can walk out the OR.

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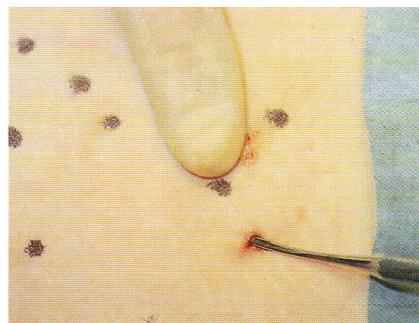
For anaesthesia – local infiltration – it is recommended to use a solution of 0,25 %, e. g. Xylonest or Meaverin. The varices become marked which facilitates preparation resp. removal.

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Tiny incision with a special micro scalpel blade followed by distal and proximal exposure of the vein by using spatula end of phlebextractor.

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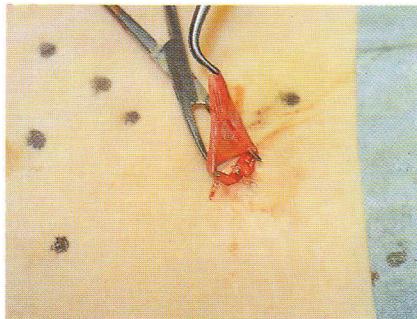
Tracing and pulling the vein out by using hook-end of the phlebextractor.

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
Grasping the vein with the Mosquito haemostatic forceps, followed then by ligating and separating.

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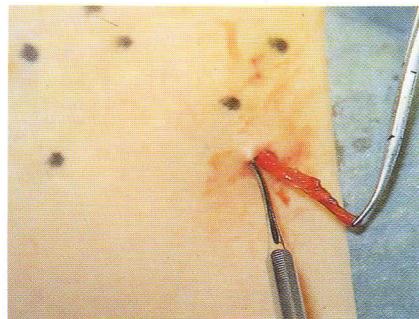
Continue preparation and under slight tension turn the vein out as far as possible.

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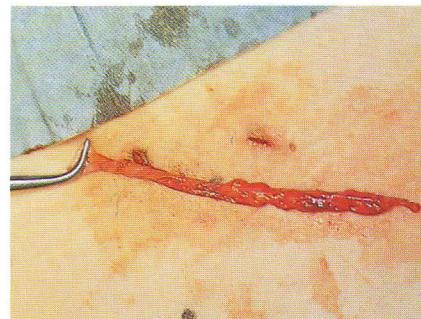At intervals of approx. 5-10 cm the opposite end is exposed after a second incision and the complete varice is removed.

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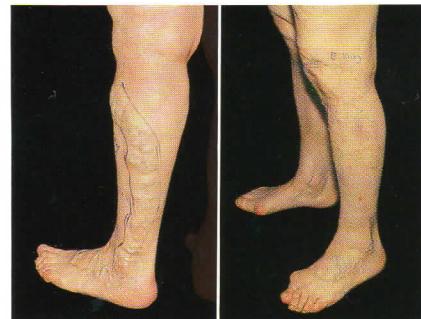
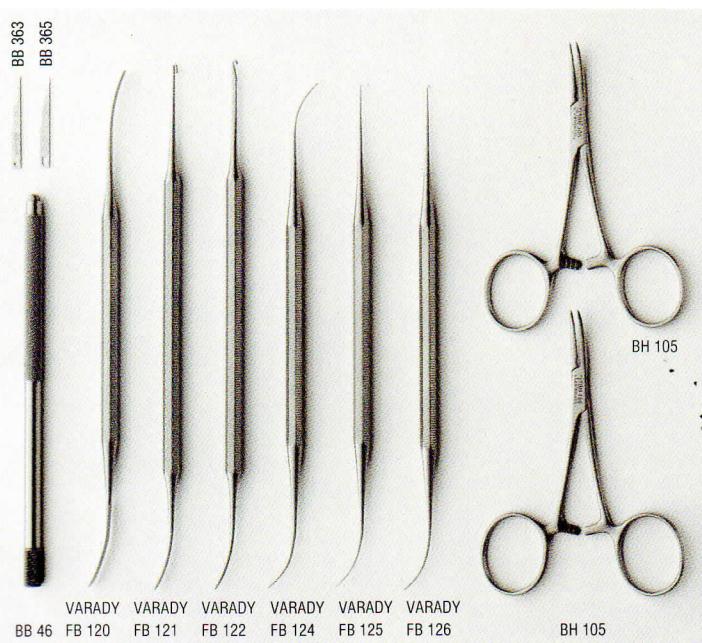
VARADY Set for Varices Surgery
consisting of:

Handle for micro scalpel blades	
VARADY Phlebodissector	
VARADY Phlebextractor	
VARADY Phlebextractor	
VARADY Mikro-spatula	
VARADY Micro-Phlebextractor	
VARADY Mini-Phlebextractor	
Flexible cable, polyethylene coating, 450 mm (17 1/2") long	
Metal olive, diam. 3 mm	
Metal olive, diam. 4 mm	
Metal olive, diam. 5 mm	
Mosquito haemostatic forceps, curved	
Small set container, with perforated lid and bottom and with long-term sterile textile filters, with ring- shaped silicon lid sealing, safety locks with plastic seal, 2 holders for identification labels	
Identification labels, inscription as desired	
Silicon cushioning pad, for safe storage of delicate instruments	

Wrapping drape

Additionally:

Microsurgery blades, stainless steel,
steril package, shape of blade and
number of pieces as desired

BB 363 R Package of 1 piece

BB 365 R Package of 10 pieces

Subject to technical modifications.

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AESCU LAP AG & CO. KG